

ISA & SIRENA Cytology application



CYTOLOGY – EZ Cytofunnel use



NOTE: The tab on the left side of the chamber must not be engaged before closing the EZ funnel



Loading the EZ cytofunnel:

1. Hold the cytofunnel in your left hand, avoiding touching the closing tab. Hold the blade holder in your right hand



2. Often the hinge by folding the two parts of the chamber several times.



3. Place the microscope slide in its housing




Closing the EZ cytofunnel:

4. Close the chamber by pressing firmly at the closing points and at the back of the chamber.

ISA & SIRENA Cytology application



CYTOLOGY – EZ Cytofunnel use

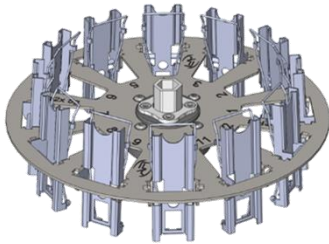

5. Position the assembly on the crown, taking care to balance the rotor, then place the necessary amount of samples on the crown.

Unloading the EZ cytofunnel:

6. After completion of the program, remove all the rotor chambers from the centrifuge.



7. Press the tab to open the blade



8. Remove the blade and throw the used cytofunnel.